Vinyl polysiloxane bite registration material
DIN 13903


FUTAR®

INSTRUCTIONS FOR USE / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing /
Инструкция по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullanım Kılavuzu /
Instruções de uso / Instrukcja użycia / Інструкція для застосування / 取扱説明書 / تعليمات الاستخدام

CONTENT

Inhalt / Contenu / Inhoud / содержимое /
Contenuto / Contenido / İçerik / Conteúdo /
Treść / Зміст / 目次 / الفهرس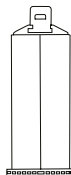

Deutsch	4
English	9
Français	13
Nederlands	18
Русский	22
Italiano	27
Español	32
Türkçe	37
Português	42
Polski	47
Українська	53
日本語	59
63	عربي

Vinyl polysiloxane bite registration material
DIN 13903

Futar[®]

Futar[®] Fast

Futar[®] D

Futar[®] D Fast

Futar[®] D Slow

Futar[®] Cut & Trim Fast

For professional use only.

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Product Description

Futar®, Futar® Fast, Futar® D, Futar® D Fast, Futar® D Slow and Futar® Cut & Trim Fast are syringeable addition-curing elastomeric bite registration materials intended for making accurate occlusal records.

Indications

All Futar® bite registration materials (Futar®, Futar® Fast, Futar® D, Futar® D Fast, Futar® D Slow, Futar® Cut & Trim Fast) can be used for making accurate occlusal registrations.

Futar® D materials (Futar® D, Futar® D Fast, Futar® D Slow, Futar® Cut & Trim Fast) can be used for applications, which require a hard bite registration silicone.

Futar® Fast, Futar® D Fast and Futar® Cut & Trim Fast can be used for “small applications” because of their shorter working time.

Futar® D Slow can be used for time-consuming bite registrations because of its longer working time.

Futar® Cut & Trim Fast can also be used for an optical registration of occlusal data for CAD/CAM systems without using powder.

Warnings

Do not use Futar® bite registration materials for conventional impressions as their high end hardness values severely inhibit withdrawal from the mouth and may result in inadvertent extraction of teeth.

Cautions

Do not use after expiration date.

Do not leave any residual bite registration material in the sulcus or oral cavity.

Do not swallow bite registration material! If swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

The product does not normally cause allergic reactions; however, for sensitive persons, an allergic reaction to the product cannot be ruled out.

Use of products containing active sulfur, aluminum chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials etc.) in conjunction with this

product will interfere with the setting reaction of the vinyl polysiloxane material. Use of these materials requires the area to be rinsed thoroughly to remove all residues.

If gloves are worn when handling the material, they should be tested with a sample of the bite registration material beforehand to ensure compatibility. Do not use latex gloves.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Note

The total working time and intraoral setting time should be strictly adhered to in order to ensure optimal bite registrations. Should the temperature of the material deviate noticeably from 23°C (74°F), the total working time and setting times will be affected.

Instructions on the Use of Cartridge Material in the Applyfix 4 Dispensing Gun DS-50 1:1/2:1

Insert the cartridge into the Applyfix 4 dispensing gun DS-50 1:1/2:1. Ensure that the notches on the cartridge base are pointing down. The clasp will not close if the cartridge is not properly inserted into the dispensing gun.

Remove the cartridge cap. The cap can be replaced after initial use.

Advance plunger into cartridge and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a mixing tip on the cartridge and turn cap 90 degrees in a clockwise direction to lock in place.

Inject material directly onto the patient's occlusal surfaces or occlusal rims.

Replace the plug after use. Before using the cartridge again, remove the cartridge cap or the filled mixing tip and check the cartridge outlet openings to ensure that no polymerized materials have sealed the openings. Clear any clogged openings and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a new mixing tip and continue as described.

Cleaning and Disinfection of the Applyfix 4

Remove the cartridge.

Remove any uncured paste using a cloth soaked in alcohol.

The Applyfix 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant.

Kettenbach recommends the use of glutaraldehyde-based solutions.

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disinfection

Bite registrations can be disinfected using for example, a 2 % acidic glutaraldehyde solution or use disinfection solutions. See manufacturer instructions.

Disposal

The empty cartridges / mixing tips / applicator tips and set material can be disposed in the household rubbish.

Trademarks

Futar® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials / products are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: January 31, 2022

Product Overview and Technical Data

Product name	Working time at 23°C/74°F ≤	Intraoral setting time at 35°C/95°F ≥	Total setting time* ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Mixing technique	Mixing element
Futar®	30 seconds	1 minute 30 seconds	2 minutes	A 90	- 0.20	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® Fast	15 seconds	45 seconds	1 minute	A 90	- 0.20	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® D	30 seconds	1 minute 30 seconds	2 minutes	D 43	- 0.10	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® D Fast	15 seconds	45 seconds	1 minute	D 43	- 0.10	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® D Slow	1 minute 30 seconds	3 minutes	4 minutes 30 seconds	D 43	- 0.10	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® Cut & Trim Fast	15 seconds	45 seconds	1 minute	D 35	- 0.20	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, yellow, Ø 4.2 mm

* Total setting time (removal time from mouth) from start of mix

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